



■ Panadent Corporation  
580 S. Rancho Avenue • Colton, California 92324, USA  
Tel: (909) 783-1841 • USA & Canada (800) 368-9777



REF 5401

# Basic Articulator Module Instructions

Introduction

Component Identification

Face-Bow (Ear-Nasion) Instructions

Maxillary Cast Mounting Instructions

Interocclusal Record Instructions

- Centric Relation
- Protrusive
- Left Lateral
- Right Lateral

Mandibular Cast Mounting Instructions

Articulator Adjustments (Protrusive)

Articulator Functions


Magna-Split II Instructions

## BASIC SYSTEM COMPONENTS



1. Articulator (Model PCH shown)
2. Articulator Support Legs (not shown)
3. **Pana-Mount**™ Face-bow
4. **Bio-Esthetic** Level Gauge
5. Molded Plastic Case
6. Mandibular Mounting Stand
7. Magna-Split II System (not shown)
8. **Bite-Tray**™ Registration Plates (20)
9. **Bite-Tab**™ Compound Discs (180)
10. Basic Instruction Manual

## FACE-BOW COMPONENTS:

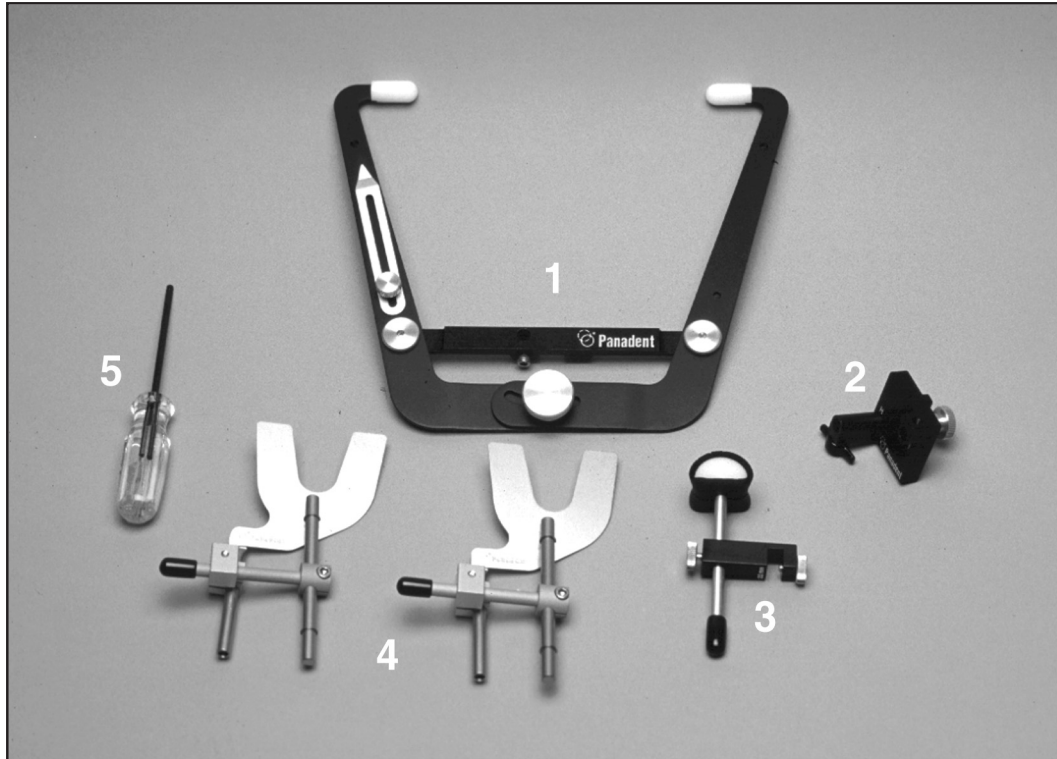

1. ***Pana-Mount*** Face-Bow (Frame Assembly)
2. Mounting Fixture
3. Nasion Relator (22mm)
4. Bite-Fork & Stem Assembly (2each)
5. Hex Wrench