




## Edentulous Outrigger Instructions

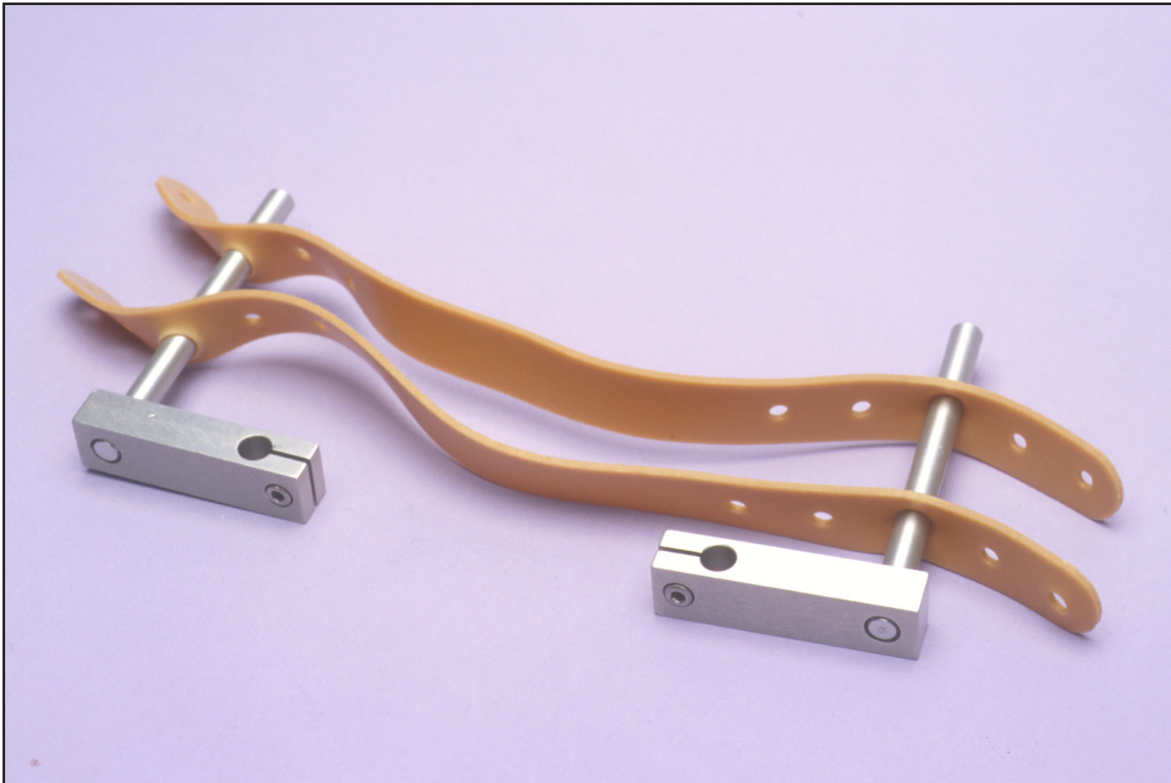


(1.) Remove patients lower denture. Leave upper denture in patients mouth. Attach plastic clutch and outriggers to cross-bar. Place highest discluser. Have outriggers 6-7 inches (depending on face width) equal distance from mid line of cross bar.

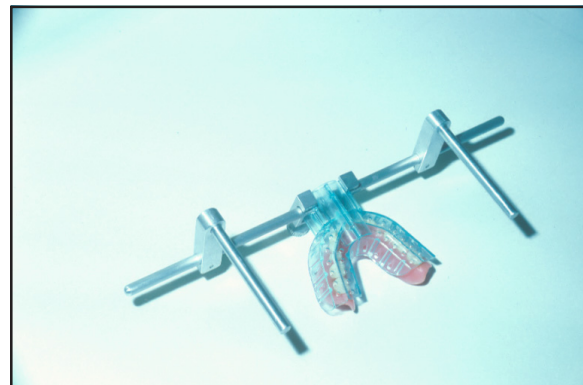

(2.) Be sure that patient's lower denture will fit easily inside clutch; if not, modify clutch.

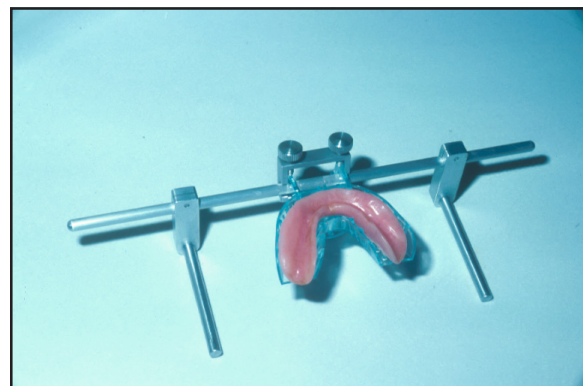


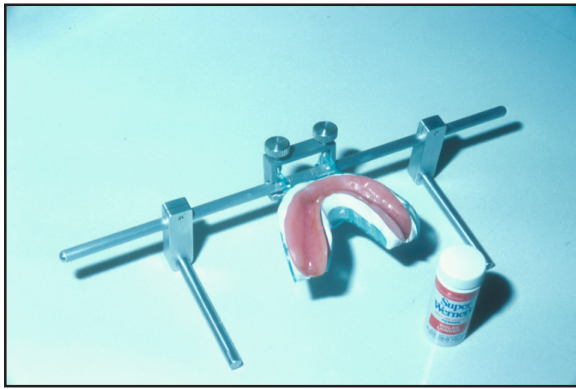

(3.) Attach lower denture to clutch with quick set plaster. Slide both elastic bands on outrigger on one side of patient only, leaving loose ends hanging free. Wet Denture and sprinkle with denture powder.



(4.) Place lower denture with cross bar attached in patients mouth, and have patient bite on two cotton rolls (placed bilaterally). While patient bites firmly on cotton rolls, stretch elastics from outrigger on one side under mandible and attach to outrigger on other side of patient (one band near chin, other band near throat). Remove maxillary denture and allow discluser of clutch to slide against anterior edentulous ridge.

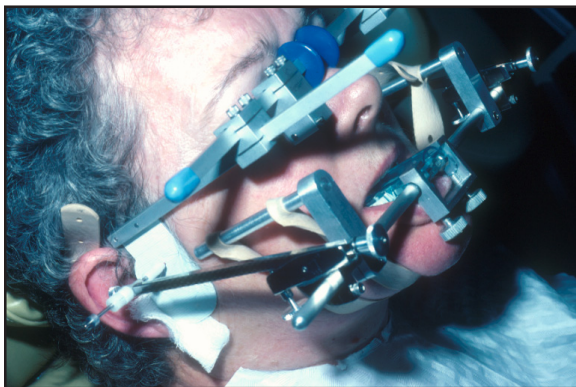

(5.) Place maxillary head frame to locate axis and record hinge axis in usual manner, then record protrusive and lateral Bennett movement. Remove head frame and mark hinge axis on skin. Replace upper denture and have patient bite on two cotton rolls. While patient is biting on cotton rolls, release elastics on one side of patient first and then remove lower denture-clutch cross-bar assembly.