

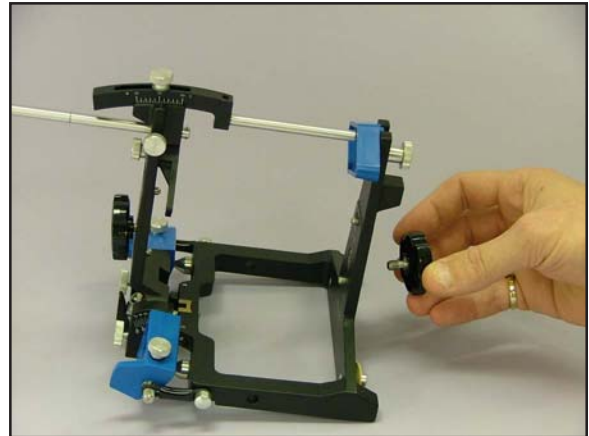
This instruction applies to the following item:

Mounting Plate Spacer Assembly Instructions

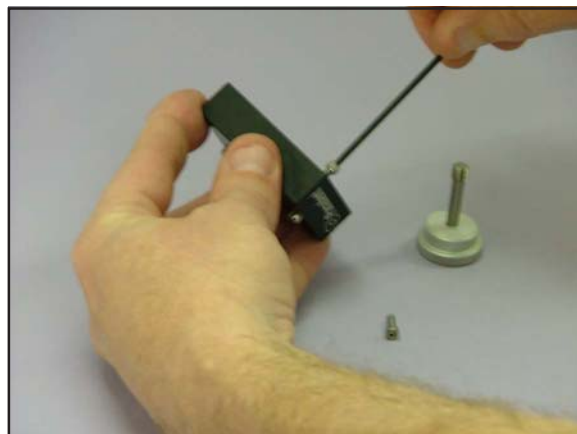
REF 2479

1

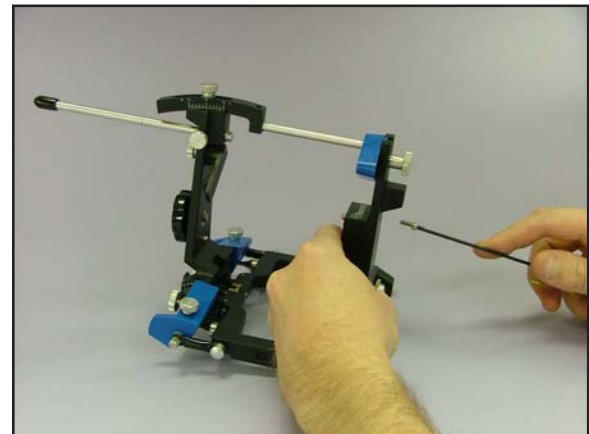
Picture of complete Mandibular Mounting Plate Spacer Assembly.

2

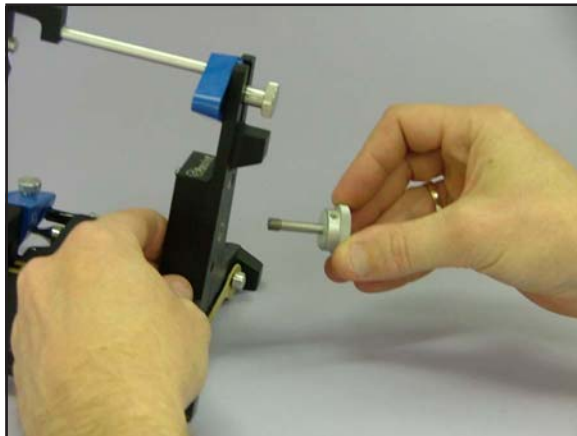
Remove standard mounting plate screw by unthreading it through the lower frame of articulator.

3

Remove extended mounting plate screw and small fastening screws from spacer assembly.

4

Attach spacer assembly onto lower frame of articulator with small fastening screws.

5

Place extended mounting plate screw by threading through the frame of articulator and spacer assembly.

6

Picture showing complete mounting plate spacer assembly on articulator.