

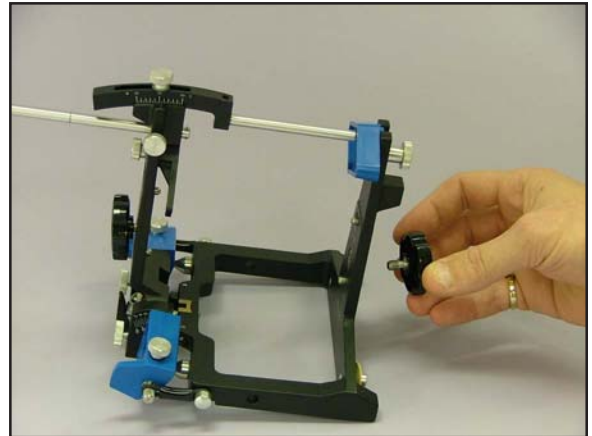
This instruction applies to the following item:

## Mounting Plate Spacer Assembly Instructions

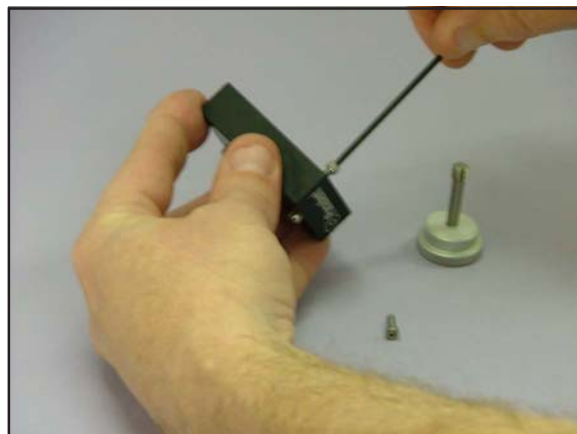
REF 2479

**1**

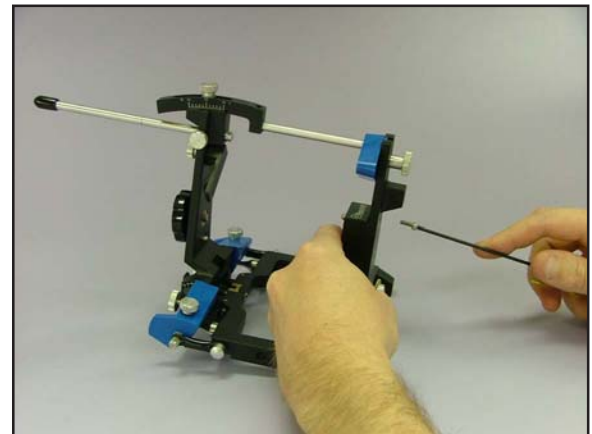
Picture of complete Mandibular Mounting Plate Spacer Assembly.

**2**

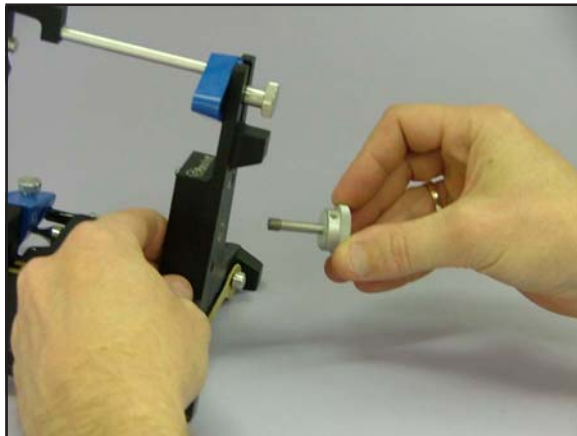
Remove standard mounting plate screw by unthreading it through the lower frame of articulator.

**3**

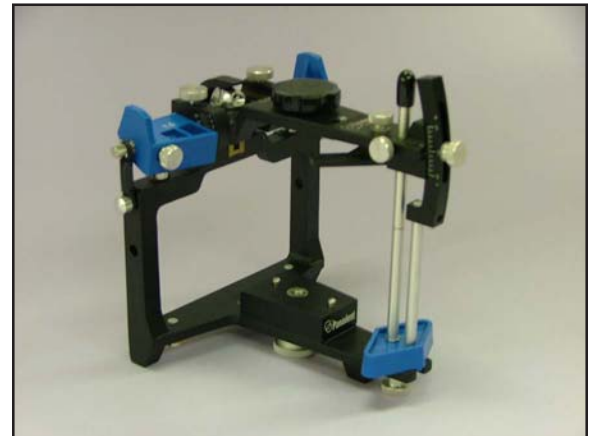
Remove extended mounting plate screw and small fastening screws from spacer assembly.

**4**

Attach spacer assembly onto lower frame of articulator with small fastening screws.

**5**

Place extended mounting plate screw by threading through the frame of articulator and spacer assembly.

**6**

Picture showing complete mounting plate spacer assembly on articulator.